

english

**B Safety instructions**

- If you have had a mastectomy, do not measure blood pressure on the side of the body where the breast has been removed together with the lymph nodes from the armpit.
- Measuring your own blood pressure does not constitute treatment. On no account modify of your own accord the dosage of drugs/medication prescribed by your doctor.
- Please refer to the chapter "Important instructions for use" (Page EN-40) before conducting any measurements yourself.

**2. Important technical details**

- A consistently good power supply to the unit is essential for trouble-free blood pressure measurements.
  - Please use long-life alkaline batteries only (LR6).
  - When replacing batteries, always replace all the batteries at the same time.
  - You will need 4 x 1.5 V AA/LR6 batteries. Rechargeable batteries have a voltage of only 1.2 V and are thus not suitable.
- When using the unit with a mains adapter, please use only the type A1 mains adapter specially tested for medical devices.
  - Make sure when operating the unit with the mains adapter that the unit can be disconnected from the mains power supply at all times.
  - If the unit is not used for an extended period of time, please remove the batteries. Essentially, all batteries can leak.
- If the unit has been stored at temperatures below 0°C, let it stand at room temperature for at least 1 hour before use.

EN-36

english

**Safety instructions B**

- The unit must only be operated with original parts. The warranty will be invalidated if the unit is damaged by non-approved accessories!
- Using the unit in the immediate vicinity of mobile phones, microwave appliances or other devices with strong electromagnetic fields may result in impaired functioning. Maintain a minimum distance of 3 m from such devices when using this unit.
- The display of the pulse frequency is not suitable for checking the frequency of cardiac pacemakers. Cardiac pacemakers and blood pressure monitors do not influence each other in their mode of operation.
- Never open or modify the unit or the cuff - this is a medical device and may only be opened by authorised specialists. (Exception: replacing batteries). If the unit has been opened, it must be subjected to a metrological inspection by an authorised institution.
- The cuff may be inflated on the arm only.
- Please comply with the specified operating conditions for measurement. See Technical data, Page EN-55.
- The inflating and measuring procedure can be stopped by pressing the Start/Stop button. In this case, the unit stops the inflation procedure and deflates the cuff.

EN-37

DOWNLOAD: <https://timurli.com/29wybr>

**Download**

7c2bc87b77

Related links:

[Tetris Zone- Precracked No Survey No Password No Download](#)  
[download film peccato veniale 1974](#)  
[Woodkid - Iron \(2011\) FLAC \\\ TOP \\\](#)  
[Avast Free License](#)  
[Devi 1999 Telugu Full Movie Free Download](#)  
  
[the Hindustani 2 in hindi download torrent](#)  
[Mission Terrain DII Silent Hunter 5](#)  
[Denim Day promotes sexual violence education and prevention](#)  
[Port Royale 2 Impero E Pirati Download Italy Music](#)  
[Say Yes To Love 2 Download Movie Free hunter.configure.vor](#)